RESCUE SAM

Choose Italian Quality

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Defibrillator

RESCUE SAM is a semi-automatic external defibrillator designed for providing simple and direct voice prompts and indications for a straightforward rescue operation. It is lightweight and battery powered for maximum portability. RESCUE SAM is designed to treat Ventricular Fibrillation (VF) and Fast Ventricular Tachycardia (VT).

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progetti
Medical Equipment Solutions

www.progettimedical.com

RESCUE SAM

TECHNICAL SPECIFICATION

OPERATING MODE: semi-automatic
WAVEFORM: BTE (biphasic truncated exponential type);
impedance compensated
ENERGY: 200J fixed as new norms IEC 60601-2-4
- adult: nominal 200 J into a 50 Ω load
- infant/child: nominal 50 joules into a 50 Ω load
SHOCK ENERGY ACCURACY:
According to IEC EN60601-2-4
SHOCK-TO-SHOCK CYCLE TIME: typically less than 20 seconds
PROTOCOL: voice prompts and indicators guide user through protocol
CHARGE CONTROL: automatic by software (arrhythmia detection system and charging control)
CHARGE TIME FROM "SHOCK ADVISED":
<8 seconds, typical
VOICE INSTRUCTIONS: detailed voice messages guide responder through use of the defibrillator
CONTROLS: shock button, info-button, On/Off button
INDICATORS: LEDs (different colors), info-button
CHARGE COMPLETE INDICATOR:
- Text prompt (press the shock red button)
- Flashing red shock button
DISARM: once charged, the RESCUE SAM disarms if:
- Patient's heart rhythm changes to non-shockable rhythm, or
- The SHOCK button is not pressed within 15 seconds after the is armed, or
- The Power ON/OFF button is pressed to turn OFF, or
- The defibrillator pads are removed from the patient or the pads connector is disconnected
SHOCK DELIVERY: shock is delivered if the SHOCK button is pressed while the defibrillator is armed
SHOCK DELIVERY VECTOR: via adult defibrillator pads in the anterior-anterior (Lead II) position or via reduced-energy pediatric pads in the anterior-posterior position
PATIENT ISOLATION: type BF
BATTERY: 12 Volt DC, 4.2 Ah, lithium manganese dioxide, disposable long-life
BATTERY CAPACITY: min.200 shocks or 4 hours of operating time continuous
AUTOMATED AND USER-ACTIVATED SELF-TESTS:
- Daily Self-tests: tests internal circuitry, waveform delivery system, battery capacity, and software.
- Battery Insertion Test: upon battery insertion, extensive automatic self-tests and user-interactive test check device readiness.
DATA RECORDING AND TRANSMISSION:
- Optional special battery pack for data transmission
- Data stored: first 60 minutes of ECG and the entire incident's events and analysis decision.
SIZE: 80x300x270 mm
WEIGHT: 2.2 kg with battery pack installed.

Standard set:

- main unit
- disposable defibrillator pads adult
- disposable battery pack

Optionals:

- preconnected defibrillator pads adult/pediatric
- disposable defibrillator pads pediatric
- carrying bag
- wall mounted cabinet
- software with USB cable adapter for PC data management

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